Prophy Solutions Compendium

Cleanic[®]

GB / D / F

P3185-LG

KerrHawe SA
6934 Bioggio/Switzerland
www.kerrhawe.com
00800 41 05 05 05

Introduction GB

At recall sessions, in the course of the ordinary «individual prophylaxis», dentists first remove calcified deposits with manual scalers and/or ultrasonic devices, then clean and polish dental surfaces with rotating brushes or rubber cups and a «prophy paste». Films and stainings, which can sometimes be rather persistent and are caused by exogenous pigments, i.e. food and coffee, tea or tobacco, have been removed so far with abrasive or highly abrasive cleaning pastes: Such pastes consisting of combined abrasives have been shown to damage the hard substance of the tooth. These scratches in the dentin and enamel have to be levelled with fine grained polishing pastes. Such cleaning systems require two to three steps until polishing is achieved, i.e. two to three pastes with different abrasive characteristics. The procedures described are time consuming and obviously lead to cumulative substance losses. This method is harmful for two reasons. First, the surfaces

of teeth and fillings remain rough which promotes the accumulation of plaque, stainings and formation of calculus. The need for professional tooth cleaning is thus artificially increased. Second, the loss of dental hard substance induced by highly abrasive prophy pastes is detrimental, especially to exposed root cement and dentin areas and may cause irritating dental sensitivities, and later even visible substance losses. Since patients in preventive care programmes retain their teeth longer these days than in former times, milder products have to be used.

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Objectives of a modern prophylaxis paste

Maximum cleaning

Both dentist and dental hygienist need a product with an efficient cleaning ability for non-mineralised deposits and stainings in a minimum of time.

Maximum polish

Polishing teeth is not only done for aesthetic reasons: Rough surfaces favour the adherence of bacteria and hence the growth of plaque.

Minimum abrasion

These days elderly men and women retain their own teeth longer than in former times; they certainly will appreciate prophylactic measures which do

not affect their dental hard substance. This aspect also emphasizes the call for a paste with a very low abrasiveness.

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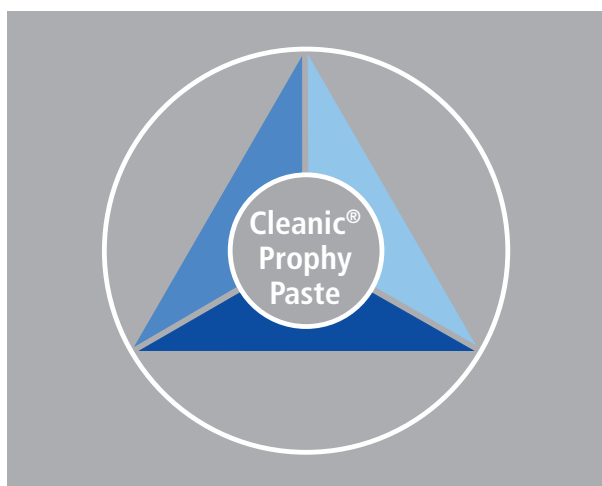
Maximum polish

Minimum abrasion

Cleanic – cleaning and polishing with one paste

Cleanic® is a universal cleaning and prophylaxis paste featuring integrated abrasion variability: The unique abrasive perlit (natural silica) converts its coarse grains to fine grains when loaded. During clinical use the fragile crystal platelets are being crushed and rounded in the rubber cup or brush by the pressure.

The cleaning ability and aggressiveness of the paste will gradually decline soon after the onset of the treatment: The cleaning paste Cleanic is converted to the polishing paste Cleanic with gloss effect.

Cleanic is a universal cleaning and prophylaxis paste for dental hard substances which may also be used on all denture materials.

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
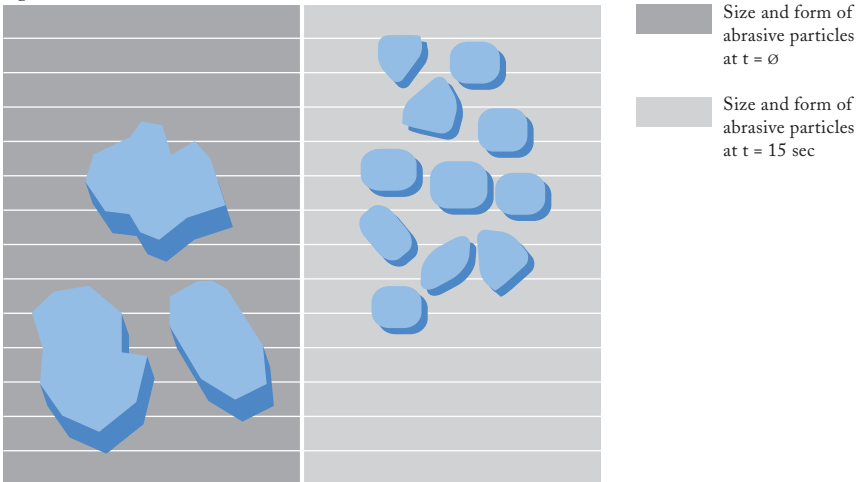
Fig. 2

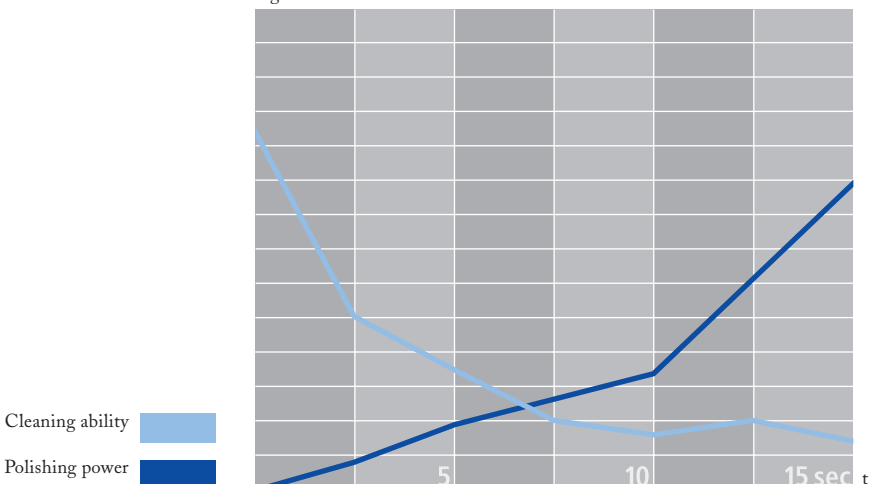Cleaning ability

Objective: Maximum cleaning ability in a minimum of time.

The chart shows the results of an invitro study: Removal of artificially produced films on human enamel with various pastes and rubber cups as a function of time.

Result: Cleanic® is shown to be at least as effective as highly abrasive pastes and clearly superior to mild pastes.

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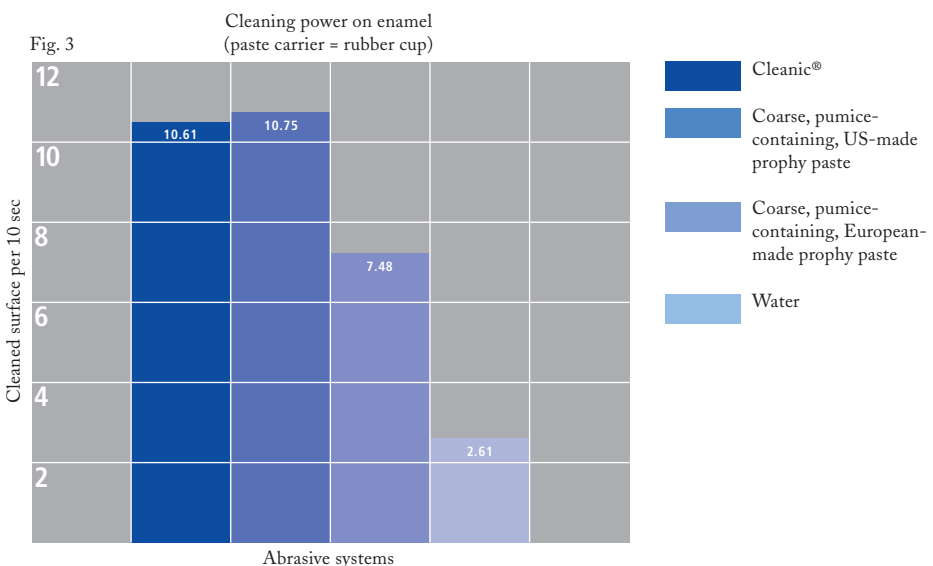
Polish and gloss effect

Objective: Maximum polish with minimal roughening of dental surfaces.

The influence of cleaning pastes and tools used (brush or rubber cup) on the surface roughness can be determined by the Ra values. This value indicates the «mean surface roughness» measured by a mechanic scanner.

Result: Cleanic® does not roughen the enamel surface despite its potent cleaning ability, whereas pumice or pastes containing pumice considerably increase the roughness (the corresponding dentin values are similar).

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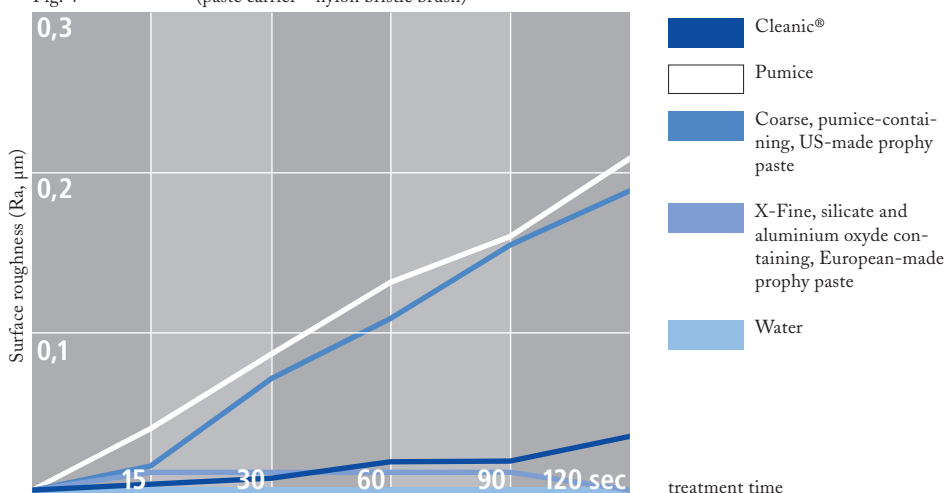
Fig. 4 Increase in surface roughness on enamel (paste carrier = nylon bristle brush)

Dentin abrasion

Objective: Minimal dentin abrasion with maximum cleaning ability.

Objectifying abrasion values in order to compare individual products is often quite difficult and timeconsuming. The procedure predominantly used these days determines the amount of dentin or enamel abraded (see: 6 enamel abrasion) within a certain time by means of radioisotopes.

RDA value = Relative Dentin Abrasion

Result: Compared to three frequently used prophylaxis pastes, Cleanic® clearly exhibits the lowest dentin abrasion values.

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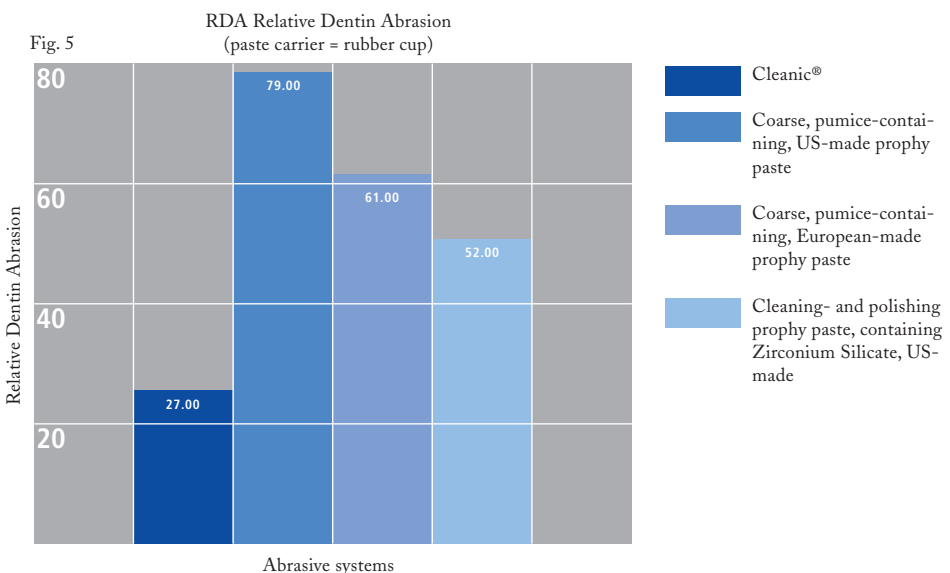
Enamel abrasion

Objective: Minimal enamel abrasion with maximum cleaning ability.

REA value = Relative Enamel Abrasion

Result: Cleanic®'s enamel abrasion values are significantly lower than those of competitive products: Despite its potent cleaning ability, Cleanic preserves the enamel.

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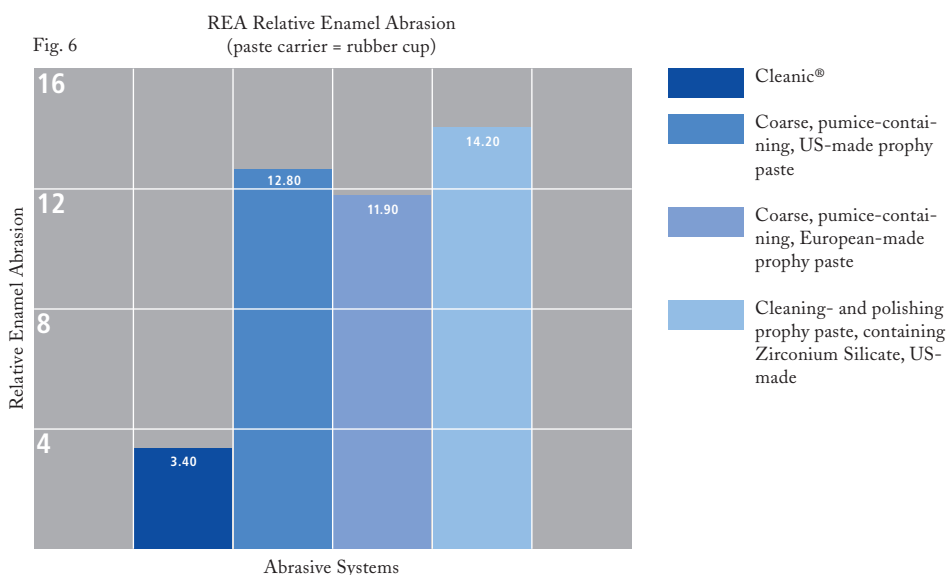
Prophy Pastes Index (PPI)

The prophy pastes' index PPI is based on the parameters of the cleaning ability, the surface roughness and the relative dentine and enamel abrasion (RDA/REA).

As from previous research, the values of the index are different according to the use of the brush or the cup. High PPI values mean high cleaning ability, reduced abrasion of the tooth substances, which results in a smooth surface.

Result: Cleanic® has the best values both using the brush and the cup and is therefore superior to the other known prophy pastes.

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
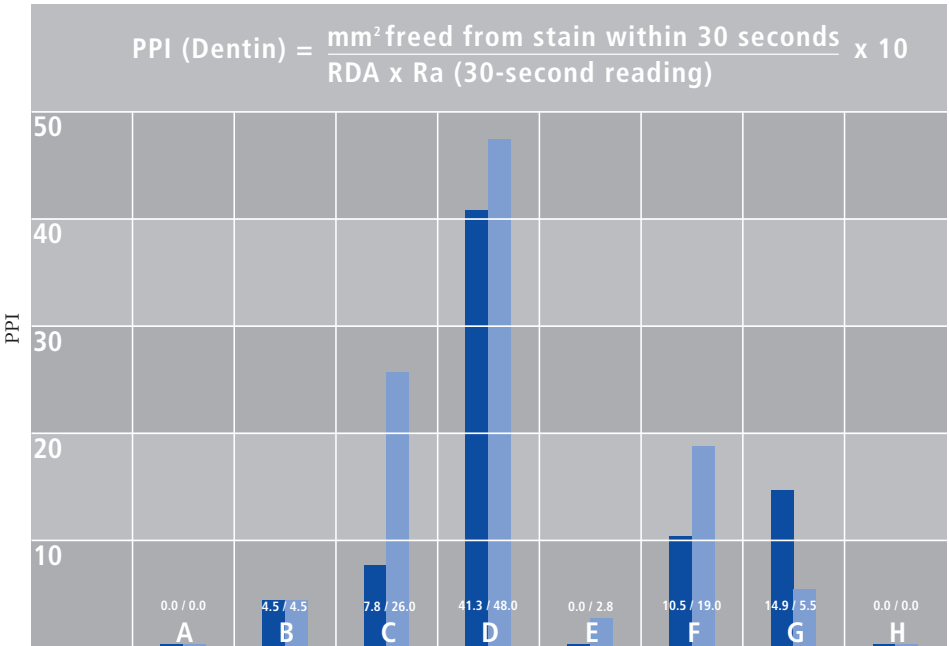
Prophy Paste Indexes – Dentin

- Rubber Cup
- Nylon Bristle Brush

- A** Water
- B** Fluor of pumice
- C** Coarse, pumice-containing, European-made prophy paste
- D** Cleanic®
- E** Tartar removal paste, containing Zirconium Silicate, French-made
- F** Coarse, pumice-containing, US-made prophy paste
- G** Cleaning- and polishing prophy paste, containing Zirconium Silicate, US-made
- H** X-Fine, silicate and aluminium oxide containing, European-made prophy paste

Prophy Paste Indexes – Enamel

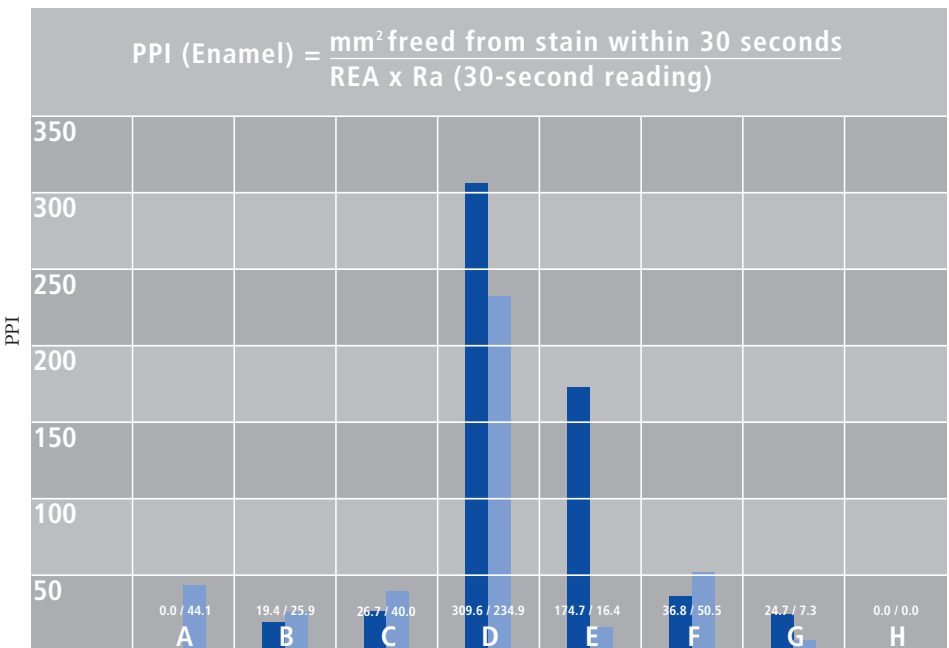Important instructions when using Cleanic

The specific paste characteristics (integrated abrasion variability, see chapter 2) require certain re-training efforts compared to conventional systems:

- Treat a limited group of teeth (about 3) with one portion of Cleanic® with a rubber cup or brush in one step from basic cleaning to polishing.
- Take a new portion and treat the next group of teeth accordingly.
- **Any residual films on the surface of a tooth should be post-treated with a new portion of Cleanic.**

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Presentation

NEW: Cleanic® is obtainable in 3 different types of packaging.

A Cleanic® Single Dose

The single dose combines maximum convenience and hygiene. No preparatory work is needed and the portion has been calculated for one treatment session.

B Cleanic® Cartridge

Contains 200 g of Cleanic® prophylaxis paste with fluoride, peppermint flavour.

NEW:

C Cleanic in Tube

Contains 100 g of Cleanic® prophylaxis paste with fluoride, peppermint flavour.

Cleanic®
US Pat. 6 576 225
US Pat. 5 266 304
US Pat. 5 124 143
EP Pat. 0 268 763
EP Pat. 0 528 756
EP Pat. 1 051 962
JP 3478549
Other Pat. Pend.

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Art. 3100

Composition

Silicates, Humectants, Binders, Surfactants, Flavouring substances (Peppermint, Menthol), Colouring Substances (CI 77346, CI 77891, CI 73015), Sweetening Substances (Saccharine Sodium or Sorbitol), Methylparabene.
Contains 0.1% Fluoride as Sodium Fluoride or 0.15% Fluoride as Calcium Fluoride.

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Bases scientifiques

- Lutz F, Sener B, Imfeld T, Barbakow F, Schüpbach P:
Comparison of the efficacy of prophylaxis pastes with conventional abrasives or a new self-adjusting abrasive. Quintessence Int 1993; 24(3): 193-201

Ordering Numbers

Order no. 3140
Cleanic® Single Dose
 200 x 2 g Single Doses
 with fluoride, 3 ring pots

Order no. 985
Ring pots
 100 ring pots

Order no. 3100
Cleanic Intro-Kit with fluoride
 1 Cleanic® container
 1 Cleanic cartridge containing
 200 g Cleanic prophy paste,
 peppermint flavour
 8 Cleanic Prophy- Clips
 3 Spatulas

Order no. 3200
Cleanic Intro-Kit
without fluoride
 1 Cleanic® container
 1 Cleanic cartridge containing
 200 g Cleanic prophy paste,
 peppermint flavour
 8 Cleanic Prophy- Clips
 3 Spatulas

NEW: Order no. 3180
Cleanic in Tube with fluoride
 100 g Cleanic® prophy paste,
 peppermint flavour

Order no. 3110
Cleanic refill cartridge with
fluoride
 200 g Cleanic® prophy paste,
 peppermint flavour

Order no. 3210
Cleanic refill cartridge without
fluoride
 200 g Cleanic® prophy paste,
 peppermint flavour

Order no. 3150
Cleanic Prophy-Clip
refill package
 30 Cleanic® Prophy-Clips

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The Pro-Cup Range

The perfect accompaniment to Cleanic® Prophy Paste.

The Problem

Traditional applicators (cups / brushes) have a tendency to splash when the paste is being applied. As the applicator rotates, the tooth may also heat up, leading to unpleasant sensations for the patient.

The solution, thanks to highly sophisticated technology

With its specially designed laminated profile Pro-Cup «propels» paste and saliva towards the edge of the cup on a helical principle. This avoids the formation of splash balls (a mixture of paste and saliva). The paste is continually forced towards the teeth. And the choice of material with a low coefficient of friction prevents teeth from overheating.

Advantages

- Hygienic, splash-free application
- Efficient use of paste
- Little overheating of teeth
- Pro-Cups soft + hard latex-free
- Smaller Pro-Cups for children and where oral conditions are restricted otherwise - Pro-Cup Junior

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