A guide  
for using

**PREMISE™**  
*flowable*

TRIMODAL COMPOSITE  
TECHNOLOGY

# **PREMISE** *flowable*

Tri-Modal Composite Restorative  
Composite de restauration trimodal  
Composite restaurador trimodal  
Trimodales Komposit für Restaurationen  
Composito tri-modale per restauri estetici  
Tri-Modalt komposit fyldningsmateriale  
Universalkomposit med tre olika typer av fillerpartiklar  
Σύνθετο υλικό αποκατάστασης τριών τύπων  
Trimodale restauratieve composiet  
Tri-Modal- komposittrestaureringsmateriale  
Kolmimodaalinen yhdistelmämuovipaikka  
Resina composta trimodal para restaurações  
Kompozytowy materiał tri-modal do wypełnień  
Трёхмодальный нанокомпозит  
Material compozit restaurativ trimodal  
Három módozatú kompozit restaurációs anyag  
Trimodalno kompozitno sredstvo za obnavljanje  
Třísložkový kompozitní materiál  
Tri-Modal Kompozit Restorasyon Materyali  
Kolmedimensiooniline komposiitaidismaterjal  
Trimodāls stiprinošs kompozītmateriāls  
Kompozicinė plomba su trijų rūšių užpildu  
Trojzložkový kompozitný výplňový materiál  
مركب علاجي للامتنان بثلاثة أشكال

# **PREMISE** *flowable*

## **Tri-Modal Composite Restorative**

Premise Flowable is a medium viscosity light cure nano filled resin restorative suitable for Class II (base/liner), Class III, and Class V restorations. This nano composite technology incorporates three different fillers resulting in a flowable, yet sculptable and non-slumping material. Additional indications include: pit and fissure sealant, repair of enamel defects, repair of porcelain restorations, minor occlusal build-ups in non-stress bearing areas, and incisal abrasions.

### **Directions for Use**

1. Isolate area using the KerrHawe OptiDam™. Dry preparation.
2. Apply a bonding system for either total etch or self-etch using OptiBond Solo Plus.
3. Select the appropriate shade of composite. Dispense a layer of Premise Flowable directly onto the preparation along the axial and gingival margins. Use a brushing action to allow the material to flow into the marginal interfaces and recesses of the preparation.
4. Incremental layers of Premise Flowable should be no more than 2 mm. Light cure each increment\*.
5. Build contours and ridges to desired shape by flowing the composite into the preparation with a brush-like motion.
6. Finish and polish using KerrHawe OptiDisc, Occlubrush or the HiLuster polishing system.

### **Shading System**

Premise Flowable is available in 10 shades that match the Premise Shade Guide (based on the Vita System) as follows:

- A1 - A2 - A3 - A3.5
- B1 - B2 - C2
- XL 1 - XL 2
- Universal Opaque

**Notes:**

1. Replace syringe cap after each use to prevent the resin from hardening in the syringe.
2. Premise Flowable dispensing tips are intended for single patient use only.

**Storage Conditions:**

Store at ambient temperature.

**\*Recommended Cure Times:** L.E.Demetron II, 5 seconds; L.E.Demetron I, 10 seconds; Optilux 501 in Boost mode 10 seconds, Ramp Mode 20 seconds or Regular Mode 20 seconds. For all other lights see manufacturer's recommendation.

**CAUTION:** Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

**LIMITED WARRANTY****LIMITATION OF KERR'S LIABILITY**

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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Kerr Italia, SpA